


# BRITEX INSTALLATION NOTES

Installation & Trouble Shooting Guide for the:

## AR-100 Elsa Arm Rest

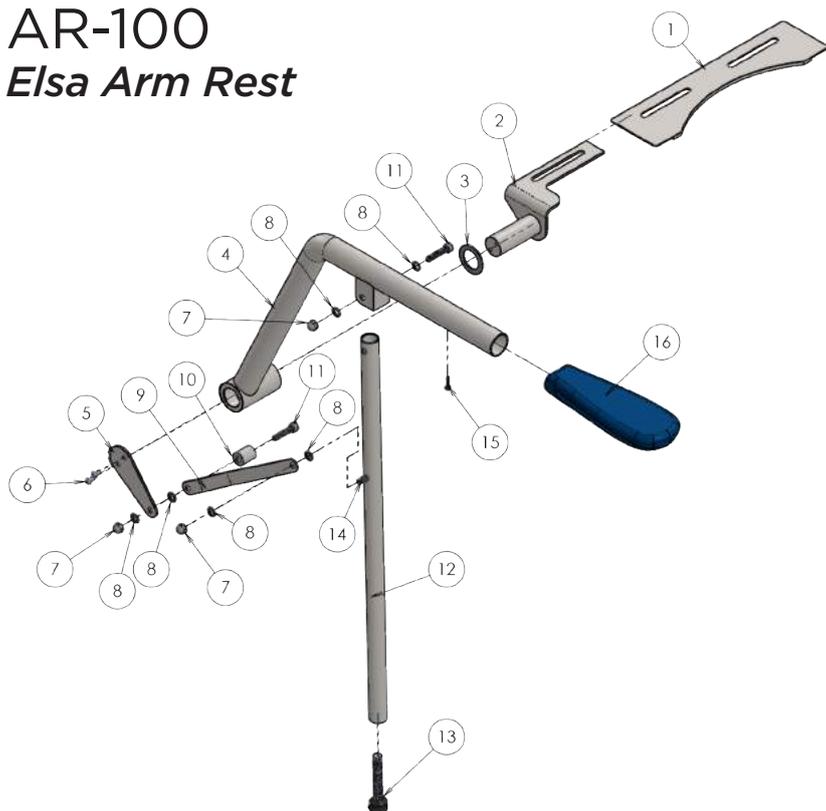

ITEM#	Description	Notes	Qty
1	Base Plate		1
2	Shoulder Bracket	1 per arm	2
3	30mm Washer	1 per arm	2
4	Arm	Left x 1 Right x 1	2
5	Pivot Mount	1 per arm	2
6	M5 Button Head 20mm Screws		4
7	M8 Dome Nut	3 per arm	6
8	M8 Washer	6 per arm	12
9	Leg Support Bracket	1 per arm	2
10	Bump Stop	1 per arm	2
11	35mm Screw	2 per arm	4
12	Leg	1 per arm	2
13	Foot	1 per arm	2
14	Stud	Attached to Leg	2
15	M3 button Head Screw for Arm Pad	1 per arm	2
16	Blue Arm Pad	1 per arm	2

\*Included but not shown: Optional Removable Toilet Roll Holder and Additional Toilet Seat Bolts

### STEP BY STEP INSTALLATION

Note: (#) Below correlate to item# above.

**STEP 1:** Pre Assembly - Place (3) 30mm Washer onto (2) Shoulder Bracket then add (4) Arm. Align (5) Pivot Mount and attach onto shoulder bracket and arm piece with 2 x (6) Button Head Screws.

**STEP 2:** Attach the (12) Leg to (4) Arm with (11) 35mm Screw & (8) M8 Washer from the inside and fasten with another (8) M8 Washer and (7) Dome Nut.

**STEP 3:** To Attach (9) Leg Support Bracket to (5) Pivot Mount, place 1 x (8) M8 Washer in between then on the inside thread (11) 35mm Screw through (10) Bump Stop and through the (9) Leg Support Bracket and (5) Pivot Mount. Fix with 1x (8) M8 Washer (7) Dome Nut to the outside of (5) Pivot Mount. Attach the other end of the (9) Leg Support Bracket to the (14) Stud (attached to the (12) Leg) with (8) M8 Washer in-between joint and add external (8) M8 Washer and fasten with (7) Dome Nut.

**STEP 4:** Push Blue Arm Pad onto (2) Arm, align fixing holes and secure with (15) M3 Button Head Screw for Arm Pad.

**STEP 5:** Repeat the Steps for opposite Arm Rest.


**STEP 6:** Remove Toilet Seat, processes may be different depending on your style. (Additional complimentary toilet seat extension bolts are included with your arm rest to help with re-fixing your toilet seat)

**STEP 7:** Place (1) Base Plate on top of Pan. Slide (2) Shoulder Bracket underneath and align slots. Fasten with Toilet Seat Brackets and re-fix Toilet Seat Fixings. Keep slightly loose until repeated for other side, then tighten all fixings.

**STEP 8:** Fasten Toilet Seat then adjust (13) Feet on the Arm Rest by screwing down to the suitable height.

**Enjoy the ease and safety of using the Elsa Arm Rest!**

Last Revised May 2018

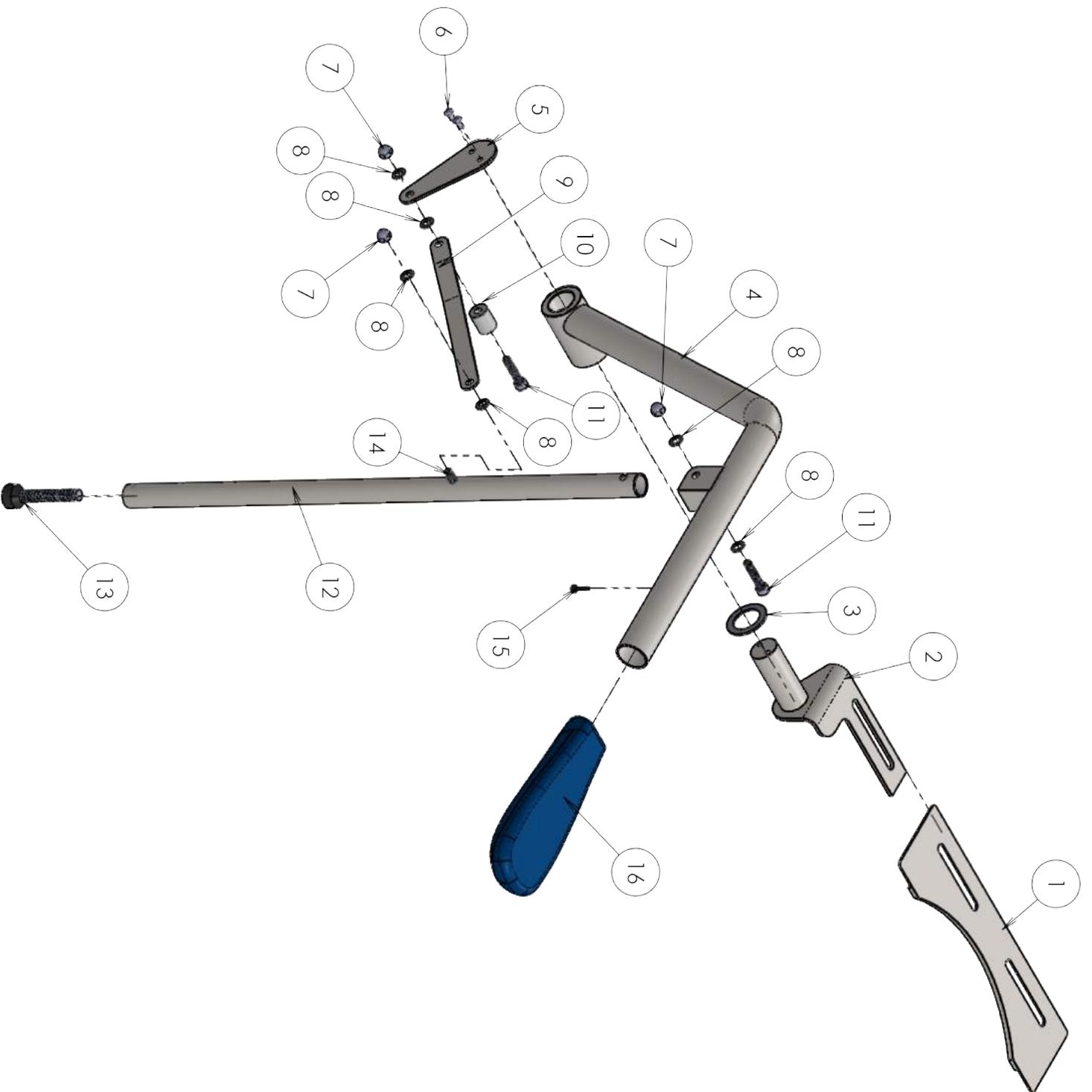

ITEM#	Description	Notes	Qty
1	Base Plate		1
2	Shoulder Bracket	1 per arm	2
3	30mm Washer	1 per arm	2
4	Arm	Left x 1 Right x 1	2
5	Pivot Mount	1 per arm	2
6	M5 Button Head 20mm Screws	2 per arm	4
7	M8 Dome Nut	3 per arm	6
8	M8 Washer	6 per arm	12
9	Leg Support Bracket	1 per arm	2
10	Bump Stop	1 per arm	2
11	35mm Screw	2 per arm	4
12	Leg	1 per arm	2
13	Foot	1 per arm	2
14	Stud	Attached to Leg	2
15	M3 button Head Screw for Arm Pad	1 per arm	2
16	Blue Arm Pad	1 per arm	2